



Product Overview

Whole Body Photobiomodulation System

Prism Light Pod is the industry's most-advanced, award-winning full body red light therapy system. We have been awarded the "Best & Most Powerful" red light bed by Lux Life Magazine for 2021 and 2020.

It's the **most powerful** (at a maximum of 100 MW/CM²) 14,000 LEDs are administered closest to the body, (half-inch from the surface) for the deepest and most effective mitochondrial-level healing.

It's the **most energy-efficient** (supports standard 110V electrical outlets with a dedicated 20-amp circuit breaker. No need to install 220V electrical package and saves thousands of dollars in ongoing energy costs compared to all other light beds requiring 220V & 30-40 amps.

It's the **most optimized & automated** with six-use case settings for one-button, 100% unattended 15-minute timed-out sessions.



Prism Light Pod Confidential – Do Not Forward Without Written Authorization

The light pod is FDA registered and administers 14,000 630nm, 660nm red and 850nm near-infrared industrial-grade 100 MW/CM² LEDs. The LEDs are closest to the body, a half inch away from the surface, with the **deepest** light wave penetration when compared to all other light beds. Other light beds' LEDs are 3-5 ½ inches away from the surface on the bottom and 24 inches away from top, **losing 80%** of the mitochondrial healing benefits.

Prism Light Pod is the only fully automated system with six optimized use-case settings for 100% “Unattended” use, so you do not have to be an expert in photobiomodulation settings. It’s a one button operation system and automatically times out after a 15-minute session.



The Prism Light Pod LEDs are rated for a 10-year life or 100,000 hours MTBF (mean-Time-Between-Failure) and includes a five-year warranty on the frame and two-year on all electronic parts. Post warranty support is available including phone and on-site technical support if needed.

When you purchase a Prism Light Pod for commercial use, it comes with our in-house Pro-Partner Joint Marketing Program. We help you build a profitable light therapy business, which may generate more than \$100,000 in incremental income per year.

The joint marketing program includes your choice of a white color light pod, co-branding of the light pod with your company logo, light therapy session pack recommended pricing and solution bundling, marketing promotional tactics, signage, pop-up banner, website & social media content, PR, and training. The Pro Partner Joint Marketing program is a \$10,000 value at no additional cost to you.

For an additional fee, our Prism Light Pod team will travel to your location to join you for a 3-hour launch event. During this time we educate your community on the benefits of Prism Light Pod and drive sales of Prism Light Pod Monthly Membership packages.

Our preferred equipment financing partners may offer you 60-month, zero-down financing for approximately \$1500/month or some money down with shorter terms if you prefer depending on your commercial business credit history.

In total, you receive **more than \$100,000 of value** including an MSRP of \$79,995 for the Prism Light Pod at a first-in-your-market price of **\$69,995 plus shipping/onsite installation and sales tax**. That is 40% more value than our closest competitors for the **BEST Award-Winning and most-advanced light pod** in the natural wellness industry.

Product Highlights:

- The Prism Light Pod is safe, comfortable and non-invasive. It is non-fluorescent, non-UV Rays and doesn't use infrared heat-based LEDs
- It accelerates your body's natural recovery process by 4 to 10 times faster by penetrating muscles, tissues, nerves, body cells and mitochondrial wellness
Prism Light Pod uses cold laser, Class II technology that is FDA registered
- Simply lay in the light pod 2-4 times per week for 15-minute sessions to see healing results after a few sessions
- 7 feet of cold laser and near-infrared light waves surround your entire body, 360 degrees from head to toe.
- Prism Light Pod is also fully-automated with six optimized irradiance settings for the following use cases that satisfy 95% of patient uses cases:
 - **Sports Rehab and Recovery:** Accelerates healing of ligaments, tendons, muscles, stiffness & soreness.
 - **Reduces Chronic Pain:** Reduces inflammation, nerve damage and autoimmune conditions.
 - **Wounds and Injury Healing:** Accelerates healing of sprains, strains, open wounds and post-surgical procedures.
 - **Skin Conditions and Anti-Aging:** Reduces wrinkles, improves skin tone and conditions, including, but not limited to Psoriasis, Acne, Eczema and Dermatitis.
 - **Weight Loss:** Lose inches by shrinking unwanted fat cells and following the Prism Fresh-Start nutrition plan & fitness guidelines.
- Our competition requires you to become a photobiomodulation expert requiring you to set the irradiance (light wave frequency & intensity) for each customer use-case and determining the length of time required for their light sessions that may vary from 8 to 20 minutes with competitors' light beds.
- Each Prism Light Pod session is 15-minutes and automated. Our Software automatically optimizes the irradiance settings for each use case and times out the session automatically.

Prism Light Pod technical dimensions & usage guidelines:

- Dimensions: 92” long X 4 feet deep X 4 feet tall (when the top lid is open).
- Fits within an 8x10 or 9x10 feet size room with a locking door.
- LED irradiance is rated at 100 milliwatts per centimeter square, closest to the body making it the most powerful within the industry and closest to the body.
- Prism Light Pod industrial-grade red LEDs last 10 years or 100,000 MTBF hours.
- Prism Light Pod uses a mix of 630nm, 650nm red light and 850nm near-infrared wavelengths with efficient power distribution that outputs 17 amps at peak for maximum electrical efficiency and reliability.
- The light Pod plugs into a standard 110V electrical outlet (no re plumbing to 220V, which saves customers \$3,000-\$5,000 compared to competitive Light Pods).
- Supports 220V/240V for international electrical requirements. •
- Prism Light Pod uses software to optimize our six settings with WIFI upgradeable support capabilities.
- Prism Light Pod is rated to support up to a 400 lb. person.
- Includes the patient intake, waiver & consent form to understand the patient’s primary use-case and assess their results over time.
- Eye protection wear is optional, and a pair of goggles comes with the Light Pod. Wearing eye protection is recommended when using the light pod.
- The Prism Light Pod therapy is “unattended” (Does not require a doctor or technician to purchase, administer, nor be present for the therapy). Users simply select their primary use-case, press start and the Light Pod illuminates for a 15-minute session and the LEDs automatically shut off when time is up.
- Wipe down the Light Pod with disinfectant spray or Lysol/Clorox antiseptic wipes and it’s ready for the next appointment.
- User appointments should be booked at 20 minutes intervals (5 mins to change, 15 mins to lay in the Light Pod.)
- Cover-up any other part(s) of your body that you don’t want to receive the red light therapy, such as wearing a sports bra if you prefer not to lose fat cells there.

- **Contraindications:** Patients who are pregnant should not use the Prism Light Pod. Women who are breastfeeding may see a reduction in producing lactose. Patients with active cancer must consult their physician and bring a Doctor's permission note first before using the light pod. Always refer patients to their physician if they have any concerns.
- Always have patients sign the Waiver and Consent Form prior to the light pod treatment.
- Prism Light Pod is 100% designed and manufactured within the United States, headquartered in Denver, Colorado.
- The Prism Light Pod system includes a five-year warranty on the frame and a two-year all parts warranty on all electronic parts with phone support Monday - Saturday, from 8 am to 5 pm, Mountain Standard time. The light pod is designed for self-repair saving you repair costs. Our technicians will conduct troubleshooting FaceTime and Zoom calls to assist you with any repairs or replacement parts.
- We do sell post-warranty onsite repair services at \$180/hour of labor, travel, time and materials if needed.

Join the fast-growing photobiomodulation natural-healing movement to benefit you and your customers. Generate \$100,000 or more in profits per year with our 100% unattended whole-body cold laser light pod.

Call us to learn more about Prism Light Pod and our Pro-Partner Joint Marketing program, a \$10,000 value at no additional cost for you to be profitable at day-1 launch.

For more information, please contact us at: (720) 664-4929 or info@prismlightpod.com or www.prismlightpod.com

See our patient testimonials at www.prismlightpod.com/testimonials

Read our blogs at www.prismlightpod.com/blogs

Follow us on social media

Instagram: <https://www.instagram.com/prismlightpod/>

Facebook: <https://www.facebook.com/prismlightpod/>

LinkedIn: <https://bit.ly/2Kxo2CR>

Twitter: <https://twitter.com/prismlightpod>

Write your notes and questions below to schedule a follow up call with your Prism Light Pod account representative: